

## Geratherm® daisy color With Visual Fever Color Indicator

### Instructions for Use:

1. Press the ON/OFF button. The display will read 188.8° confirming that the display is working and the Fever Color Indicator (colored backlight) will flash (red -> yellow -> green). The display will show “L” with °F flashing.
2. Place the pacifier in child’s mouth. **Note:** Drinking or eating prior to use may affect reading.
3. Once the degree sign (°F) stops flashing (approximately 3 minutes), the alarm will sound, indicating the temperature is ready to read and the Fever Color Indicator will light up for 5 seconds (see chart below).
4. The unit will automatically turn off in approximately 30 seconds. To save energy, it is best to turn off manually by pressing the ON/OFF button.
5. To Clean: Wipe nipple with soap and warm water or rubbing alcohol.
6. Memory Recall: Press and hold the ON/OFF button for more than 3 seconds. The last measured temperature will appear with the letter “M” under the °F sign.

Temperature °F	Fever Alarm Indicator
96.8—98.8	Green
98.9—99.3	Yellow
99.5—111.0	Red

## Geratherm® daisy color Con indicador de fiebre con colores

### Instrucciones de uso:

1. Presione el botón de ENCENDIDO/APAGADO. El visor mostrará 188,8°, lo que confirmará que el visor está funcionando y destellará el indicador de fiebre (luz de fondo de color) con colores (rojo -> amarillo ->verde). El visor mostrará “L” con los °F destellantes.
2. Coloque el termómetro en la boca de la criatura. **Nota:** si se bebe o come antes de tomar la temperatura, es posible que se altere el resultado.
3. Una vez que deje de destellar el símbolo de grados (°F) (aproximadamente tres minutos), sonará la alarma indicando que la temperatura está lista para su lectura y el indicador de fiebre con colores se encenderá por cinco segundos (ver cuadro a continuación).

4. La unidad se apagará automáticamente en aproximadamente 30 segundos. Para ahorrar energía, lo mejor es apagarlo de manera manual mediante el botón de ENCENDIDO/APAGADO.
5. Para limpiarlo: frote la tetina con agua tibia a jabón o con alcohol para frotar.
6. Recordar memoria: mantenga presionado el botón de ENCENDIDO/APAGADO durante aproximadamente tres segundos. Se mostrará la última temperatura tomada con la letra "M" debajo del símbolo °F.

Temperatura en °F	Indicador de la alarma de fiebre
96.8—98.8	Verde
98.9—99.3	Amarillo
99.5—111.0	Rojo