

MagnaTherm®

Developed and manufactured in Germany by Geratherm A.G. Contains Galinstan®, a patented, non-toxic, Earth friendly substitute for mercury.

Instructions for Use:

1. Grasp top of thermometer and shake with a snapping motion of the wrist. Repeat until line is well below 98.6° (37°C).
2. Place thermometer under the tongue for a minimum of 3 minutes.
3. Replace thermometer in magnifying case to view temperature.
4. Before storing, wipe clean with cotton ball or gauze square saturated with alcohol.

Desarrollado y fabricado en Alemania por Geratherm A.G. Contiene Galinstan®, un substitute de mercurio, patentado, no tóxico y que no causa daño al ambiente.

Instrucciones para su uso:

1. Sujete la parte superior del termómetro y sacúdalo con un movimiento firme de la muñeca. Repetir hasta que la línea esté por debajo de 98.6°F (37°C).
2. Colocar el termómetro debajo de la lengua por 3 minutos.
3. Coloque el termómetro en el estuche par aver la temperature.
4. Antes de guardar, limpie el termómetro con algodón o gasa saturados con alcohol.