

Mercury-Free Rectal

Rectal sin mercurio

Developed and manufactured in Germany by Geratherm A.G. Contains Galinstan[®], a patented, non-toxic, Earth friendly substitute for mercury.

Instructions for Use:

1. Grasp top of thermometer and shake with a snapping motion of the wrist. Repeat until line is well below 98.6° (35.5°C).
2. Apply a water-soluble lubricant to the tip before inserting 1" (2.5 cm). Temperature reading in a minimum of 3 minutes.
3. Disinfect with cotton ball or gauze square saturated with alcohol before and after use.

Desarrollado y fabricado en Alemania por Geratherm A.G. Contiene Galinstan[®], un substitute de mercurio, patentado, no tóxico y que no causa daño al ambiente.

Instrucciones para su uso:

1. Sujete la parte superior del termómetro y sacúdalo con un movimiento firme de la muñeca. Repetir hasta que la línea esté por debajo de 98.6°F (35.5°C).
2. Aplique en la punta un lubricante soluble en agua antes de insertar 1 pulgada (2.5 cm). La temperature se lee en un mìnimo de 3 minutos.
3. Antes de usar y de guardar, desinfecte el termómetro con algodón o gasa saturados con alcohol.